




LED Magnifying Lamp

ITEM: MSI-FCAPP-128

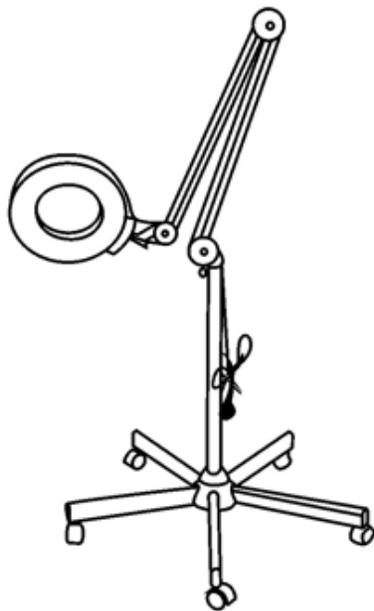

Thank you for purchasing our Dermalogic Magnifying Lamp. To use this product safely and correctly, please read this manual carefully and abide by all directions and notes.



Package Contents



Lamp x 1

Base x 1

Leg with
caster x 5

Pole x 1

Specification

Model NO.	Wattage	Voltage	Frequency
MSI-128	5W	110V	60Hz

Operation Instructions

1. Plug in the power plug of the LED lamp and move the magnifying lamp above customer's skin.
2. Turn on the power switch of the LED light and aim the magnifying glass at the skin's surface.
3. Adjust the distance between the lamp and the skin surface to achieve the clearest look.

Safety Instructions

- Place the appliance on a flat, level, and solid surface.
- Power applied on this appliance must match with the rated voltage.
- When setting up the magnifying lamp, unfold it slowly to avoid injury.
- Children are not allowed to operate this appliance for safety concerns.

Assembly Instructions

1




- Remove the bolts from each leg.
- Insert the legs onto the slots of the base.
- Screw the bolts back onto each leg.

2




Install the pole onto the base and rotate until fastened.

3

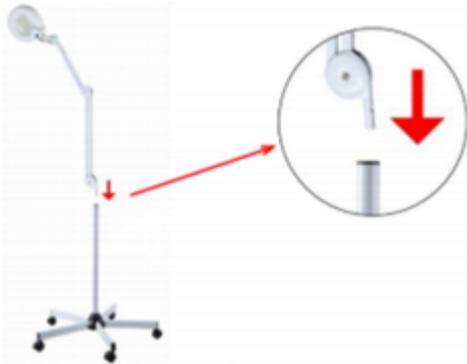

Insert the bottom end of the lamp into the top of the pole as shown.

Adjusting Magnifying Lamp

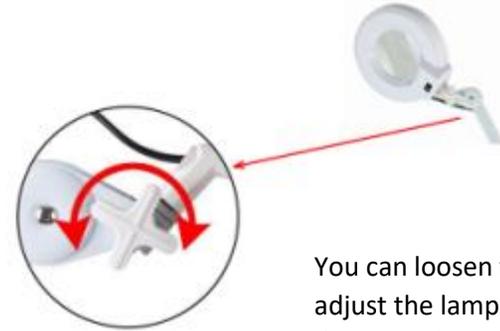

You can loosen the side knob and adjust the lamp head upward and downward. Remember to tighten the side knob after the angle is decided.



You can also adjust the lamp head to right or left and adjust the arm up and down as shown.